



## Instruction for Use

### **Product Name**

Reusable Incontinence Pad

### **Model**

40x50cm, 45x45cm, 45x60cm, 50x50cm, 50x60cm, 51x56cm, 53x56cm, 45x61cm, 75x90cm, 76x86cm, 80x90cm, 85x90cm, 85x135cm, 86x91cm, 86x132cm, 86x138cm, 86x96cm, 90x132cm, 90x140cm, 90x120cm, 90x150cm, 90x180cm, 91x91cm, 91x138cm, 91x142cm, 91x152cm, 91x182cm, 100x140cm, 110x130cm, 111x132cm All the size can be produced with or without tucks/wings

### **Device Description**

A flat piece of absorbent fabric material [e.g. polyester+ non-woven polyester with viscose + polyester with PU/PVC, or polyester + non-woven polyester with viscose + polyester with PU/PVC + cotton/polyester fabric, or polyester + padding + non-woven polyester with viscose + polyester with PU] intended to be placed on a surface (e.g., bed mattress, chair, sofa) and underneath a patient to absorb bodily fluids resulting from surgical procedures, childbirth and/or incontinence; it does not include an antimicrobial agent(s). It is intended to be used in the healthcare facility and in the home.

### **Intended Use**

The intended use of this product is for use in homes, hospitals, and nursing homes to protect the elderly and incontinent patients from urine or other body fluids seeping into beds, sofas, chairs, or other areas and to provide isolation and protection.

### **Appearance and Components**

Made from warp knit fabric, non-woven polyester, padding polyester and TPU/PVC composite or sewn. Provide non-sterile.

### **Usage**

1. Read the label and instruction for use carefully before used;
2. Take off the package, do not use the product once the package was damaged or the product has expired;
3. Put the incontinence pads on the sick bed/ chair/sofa or other surfaces that need to be protected and isolated.

## Precaution and Warning

1. This product is reusable, please timely change the clean product.
2. This product is not sterilized before delivery and is not provided aseptically.
3. Do not use it with electric blankets and other heating appliances.
4. Please keep the product out of reach of children and old people to avoid accidental ingestion.

## Contraindications

Do not use if you are allergic to warp knit fabric

## Storage

These products should be stored in a cool dry area, away from heat and direct sunlight.

## Cleaning Method

1. Do not use fabric softener wash before initial use
2. Washing temperature < 95°
3. Tumble dry at low temperature
4. Do not Dry Clean/ Do not Bleach/ Do not Iron/Do not use chemical cleaning
5. Do not steam drying or press drying

## Shelf Life

Five years

## Labels, Packing Logo Design

| Symbol                                                                              | Introductions                 | Symbol                                                                              | Introductions                                       |
|-------------------------------------------------------------------------------------|-------------------------------|-------------------------------------------------------------------------------------|-----------------------------------------------------|
|  | CE Symbol                     |  | Medical device                                      |
|  | Manufacturer name and address |  | Authorised representative in the European Community |
|  | Manufacture date              |  | Use until year & month (expiration date)            |
|  | Batch code                    |  | Do not reuse / single use                           |
|  | Warnings and precautions      |  | Non-sterile                                         |

|                                                                                   |                              |                                                                                   |                                  |
|-----------------------------------------------------------------------------------|------------------------------|-----------------------------------------------------------------------------------|----------------------------------|
|  | Consult instructions for use |  | Do not use if package is damaged |
|  | Unique device identification |  | Country of manufacture           |
|  | Importer                     |  | Distributor                      |
|  | Model number                 |                                                                                   |                                  |

## Manufacturer Information



**Company:** Changzhou Newrise Medical & Hygiene Products Co., Ltd

**Address:** No. 39 Chuangye Road, Zhulin Town, Jintan District,  
Changzhou City, Jiangsu, China 213241

## European Authorized Representative (EU REP)



**Company:** SUNGO Europe B.V.

**Address:** Olympisch Stadion 24, 1076DE Amsterdam, Netherlands