BRUGSINFORMATIONER DK

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## Gebruuchsinformation

### - Instructions for use -

Hangzhou Primecare Medical Co., Ltd.  
Room 408-409, Zancheng Center West  
Shangcheng District  
310008 Hangzhou  
PEOPLE'S REPUBLIC OF CHINA

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INSTRUCTIONS FOR USE
GB

**General Information:**  
 Dear user, please read all of the following information carefully before use. Please refer to the table for the specific product features of the individual systems.  
**Intended Purpose/Description:**  
 Urine drainage systems and bags for connection to sterile catheters. Non-sterile urine drainage systems and bags are only suitable for use with urinary sheaths and for connection to leg bags.

| Device            | Filling volume in ml | Tube length in cm | Bedside bag system | Leg bag system     | Drainage tap/valve | Tube clamp | Pump ball | Non-woven coating | Fastening   |      | Fastening   | Fastening and combination options | CE 0123                |                                                                                                                              |
|-------------------|----------------------|-------------------|--------------------|--------------------|--------------------|------------|-----------|-------------------|-------------|------|-------------|-----------------------------------|------------------------|------------------------------------------------------------------------------------------------------------------------------|
| CareFlow          |                      |                   | closed             | Number of chambers |                    |            |           | Reverse side      | Twin holder | Cord | Twin holder | Elastic band                      | Support stocking/ cuff | 1 Items suitable for combination are not included in the scope of delivery. These are available to order separately from us. |
| UB 4000 TUR       | 4000                 | 120               | x                  |                    | Tap                | x          | x         |                   | x           | x    |             |                                   |                        |                                                                                                                              |
| UB 4000           | 4000                 | 120               | x                  |                    | Tap                | x          |           |                   | x           | x    |             |                                   |                        |                                                                                                                              |
| UB 2000           | 2000                 | 120/90            | x                  |                    | Tap                | x          |           |                   | x           | x    |             |                                   |                        |                                                                                                                              |
| UB 2000A          | 2000                 | 110               |                    |                    | Tap                | x          |           |                   | x           |      |             |                                   |                        |                                                                                                                              |
| UB 2000E          | 2000                 | 90                |                    |                    | Tap                |            |           |                   |             |      | x           |                                   |                        |                                                                                                                              |
| UB 1000 Day/Night | 1000                 | 30/60/90          | x                  |                    | Tap                |            |           | x                 |             | x    |             | x                                 | x                      |                                                                                                                              |
| UB 750/750A       | 750                  | 10/30/50          |                    | 1                  | Tap                |            |           | x                 |             | x    | x           | x                                 | x                      |                                                                                                                              |
| UB 750F           | 750                  | 50                |                    | 1                  | Valve              |            |           | x                 |             | x    | x           | x                                 | x                      |                                                                                                                              |
| UB 500            | 500                  | 10/30/50          |                    | 3                  | Tap                |            |           | x                 |             | x    | x           | x                                 | x                      |                                                                                                                              |

**Indication/Scope of Application:**  
 Intended for use on patients with a recumbent indwelling bladder catheter, percutaneous renal fistula or urinary sheath for collecting the drained urine by means of a urine drainage system. The patient must be cared for in a way that is tailored to their individual needs. If the attending physician has given any previous instructions, then these must be observed.  
**Contraindication and Restriction of Use:**  
 We are not aware of any absolute contraindications. Restrictions of use may result from the clinical picture of the patient. Leg bag systems should not be used on patients with macrohematuria. Our urine drainage systems and their tubes must not come into contact with areas of the skin presenting pathological changes.  
**Application/Execution/Hygiene:**  
 Before manipulating the urine drainage systems in any way (connection or change of the system/urine drainage), please carry out hygienic hand disinfection (wash and disinfect hands). Please wear disposable gloves.  
 – Take the bag out of the sterile packaging  
 – Remove protective cap on the cone. If necessary, close non-return valve on bags at drainage tap or tilt valve (if open).  
 – Separate the existing bag system from the catheter and disinfect the catheter funnel with suitable disinfectant. Observe the reaction time.  
 – Connect the bag system to the catheter by inserting the cone into the catheter funnel.  
 – Fasten the bedside bag system to the bed frame using twin holders and cord.  
 – Leg bag: For optimal drainage, ensure that the bag is placed below the bladder.  
 – Please take care that the tubes of our urine drainage systems, especially the leg bags, are not kinked to avoid accidentally disrupting the drainage.

**Disposal:**  
 – The emptied bag system should be disposed of with the household waste in accordance with national regulations.  
**Service Life/Duration of Use:**  
 – All urine drainage systems and bags mentioned are intended for single use. If a system has been removed from its original packaging and connected to a catheter or urinary sheath, it must not be reconnected to the catheter if the items have been separated from one another. Closed urinary diversion systems should not be changed routinely at fixed intervals, but should be changed as necessary after being assessed by the physician or nursing staff, according to individual criteria.  
**Packaging/Delivery:**  
 – Each sterile item is delivered in individual packaging. Sterility is only guaranteed as long as the packaging unit is unopened and undamaged, but not longer than is indicated on the label.

**Warnings/Precautions:**  
 Please note that urine drainage systems and bags connected to an indwelling bladder catheter must not be pulled in any way in order to avoid disturbing the position of the catheter and impairing its drainage performance (RISK OF INJURY)! If the bag or medical device is defective and/or no urine enters the bag, discard it immediately and use a new one. If patients affected by conditions that require urinary diversion are fitted with our urine drainage systems or bags, appropriate care/supervision must be ensured. The same applies to small children.  
**Exclusion of Liability:**  
 In general, warranty only applies if the connection of approved medical products from GHC to the device you have in hand is carried out correctly and in accordance with the instructions for use. If you prefer to connect these to devices from other manufacturers, this is possible, but GHC assumes no liability and any warranty claims become void.  
 GHC assumes no liability whatsoever for injury or other damage resulting from use of this device other than that which is in accordance with current GHC recommendations.  
 PLEASE NOTE: All information has been issued to the best of our knowledge at the time of printing. Subject to technical changes.

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|                                                     |             |                                 |                                  |                            |                                    |
|-----------------------------------------------------|-------------|---------------------------------|----------------------------------|----------------------------|------------------------------------|
| Manufacturer                                        | Importer    | Catalogue number                | Do not resterilize               | Keep away from sunlight    | Do not re-use                      |
| Authorized representative in the European community | Use-by date | Medical device                  | Single sterile barrier system    | Keep dry                   | Consult instructions for use       |
| Date of manufacture                                 | Batch code  | Sterilized using ethylene oxide | Do not use if package is damaged | Lower limit of temperature | Contains or presence of phthalates |

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